

Colorectal cancer screening with the Cologuard test

What is the Cologuard® test?

The Cologuard test finds signs of colon cancer in your stool (poop).
The Cologuard test is a non-invasive screening. It's fast and easy to use at home.

What's inside your Cologuard collection kit?

You will receive your Cologuard collection kit by mail.



How do I use my Cologuard collection kit?

1. Unpack your kit. Read the Patient Guide inside (English and Spanish).
 - Set a calendar reminder in your phone to complete your kit within 2 weeks
2. Collect your sample when you can take it to UPS® **the same day** or **the next day**.
3. Send your sample the same day or day after you collect it.
 - Schedule a free UPS pick-up (**1-844-870-8870** or Cologuard.com/UPS).
 - Take your kit to UPS. Find stores and hours at Cologuard.com/UPS. Some stores are closed on Sundays or holidays
4. Your healthcare provider will get your result about 2 weeks after Exact Sciences Laboratory receives your sample.

Watch the [How to Use Cologuard Video](#) (in 13 languages)
Scan the QR code or go to Cologuard.com/use



Questions?

Please visit Cologuard.com or call Exact Sciences Laboratories at **1-844-870-8870**.

Cologuard Test Indications and Important Risk Information

The Cologuard test is intended to screen adults 45 years of age and older who are at average risk for colorectal cancer by detecting certain DNA markers and blood in the stool. Do not use if you have had adenomas, have inflammatory bowel disease and certain hereditary syndromes, or a personal or family history of colorectal cancer. The Cologuard test is not a replacement for colonoscopy in high risk patients. The Cologuard test performance in adults ages 45-49 is estimated based on a large clinical study of patients 50 and older. The Cologuard test result should be interpreted with caution. A positive test result does not confirm the presence of cancer. Patients with a positive test result should be referred for colonoscopy. A negative test result does not confirm the absence of cancer. Patients with a negative test result should discuss with their doctor when they need to be tested again. False positives and false negative results can occur. In a clinical study, 13% of people without cancer received a positive result (false positive) and 8% of people with cancer received a negative result (false negative). Rx only.

EXACT SCIENCES CORPORATION | 5505 Endeavor Lane, Madison, WI 53719
ExactSciences.com | ExactLabs.com | 1-844-870-8870

Cologuard is a registered trademark of Exact Sciences Corporation.

© 2024 Exact Sciences Corporation. All rights reserved. M-US-CG-03667 July 2024.

 **cologuard**[®]
by EXACT SCIENCES