

Cov khoom siv no muaj dab tsi?



Lub Thawv Xa Khoom




Thawv khaws khoom



Daim Qauv Npav Lus Qhia



Raj thiab Soj Ntsuam

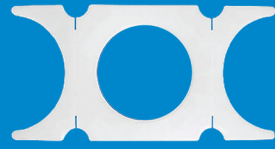

lub qhov (bracket)



Lub Hwj Ntim Cov Tshuaj Tiv Thaiv Kom Txhob Lwj

Mus saib tau rau ntawm [Cologuard.com/result](https://cologuard.com/result) txhawm rau tsim ib tus as khauj. Nyob rau ntawd, koj tuaj yeem nrhiav cov ntaub ntawv hais txog qhov kev kuaj mob uas koj tau txais thiab nkag mus saib koj cov kev kuaj tau los thaum twg muaj rau saib. Cov kev kuaj tau los raug xa mus rau tus kws muab kev kho mob uas tau sau ntawv rau koj qhov kev kuaj mob hauv 8 txog 10 hnuv tom qab tau txais koj cov qauv.

Kev Txhob Txwm Siv thiab Qhov Qhia Rau Siv

Cov khoom siv kuaj Cologuard® thiab Cologuard Plus™ yog cov kev kuaj mob zoo hauv vitro uas tsim los ntes cov cim DNA uas cuam tshuam nrog cov qog nqaij hlaw colorectal thiab ntes cov hemoglobin zais hauv cov quav tib neeg. Cov khoom siv kuaj Cologuard thiab Cologuard Plus yog tau ua nrog cov qauv uas tau sau los ntawm kev siv tag nrho cov khoom siv los cug cov quav uas Exact Sciences muab. Yog tias qhov kuaj pom tias muaj mob khees xaws hauv txoj hnyuv loj (colorectal cancer, CRC) los sis muaj mob qog nqaij hlaw ua ntej mob khees xaws loj (advanced precancerous lesions, APL), ces yuav tsum kuaj ntxiv nrog kev siv ib lub tshuab tsum saib txoj hnyuv loj. Cov khoom siv kuaj Cologuard thiab Cologuard Plus yog siv rau kev kuaj cov neeg laus hnuv nyoog 45 xyoo thiab laus dua uas muaj feem yuav mob tus mob CRC. Cov khoom siv kuaj Cologuard Plus thiab Cologuard Plus tsis tuaj yeem hloov tau qhov kev kuaj mob plab hnyuv rau kev kuaj mob los sis rau kev saib xyuas cov neeg muaj kev pheej hmoo siab.

Cov khoom siv kuaj Cologuard thiab Cologuard Plus yog ua tiav ntawm Exact Sciences Laboratories, Madison, WI.

Kev Txwv Tsis Pub Siv

Cov khoom siv kuaj Cologuard thiab Cologuard Plus tsis haum rau kev siv rau cov neeg mob uas muaj cov mob hauv qab no:

- Keeb kwm ntawm tus kheej hais txog kev mob CRC los sis APLs.
- Ib qho kev kuaj tau los pom tias muaj mob los ntawm lwm txoj hauv kev kuaj mob CRC hauv 6 lub hlis dhau los, los sis:
 - 12 lub hlis rau kev kuaj ntshav zais xyaw hauv cov quav (fecal occult blood test, FOBT) los sis kuaj tshuaj tiv thaiv kab mob hauv cov quav (fecal immunochemical test, FIT).
 - 36 lub hlis rau kev kuaj FIT-DNA.
- Tsev neeg keeb kwm muaj mob CRC txhais tau tias muaj ib tug txheeb ze ncaj qha (txiv, niam, kwv tij neej tsa, los sis menyuam) uas tau kuaj pom tias muaj mob CRC thaum muaj hnuv nyoog li cas los xij.
- Keeb kwm tus kheej uas muaj ib los sis ntau yam ntawm cov yam ua rau muaj mob CRC:
 - Kev kuaj mob ntawm cov kab mob Inflammatory Bowel Disease (mob plab hnyuv ntev, mob Crohn's)
 - Kev kuaj mob uas muaj tus mob qog nqaij hlaw hauv tsev neeg (uas yog los ntawm noob caj ces) los sis lwm yam mob polyposis, suav nrog tab sis tsis txwv rau: Kab mob qog nqaij hlaw hauv tsev neeg (FAP los sis Gardner's), Kab mob qog nqaij hlaw hauv plab uas tsis yog polyposis los ntawm noob caj ces (HNPCC los sis Lynch), Peutz-Jeghers, MYH-Associated Polyposis (MAP), Turcot's (los sis Crail's), Cowden's, Juvenile Polyposis, Cronkhite-Canada, Neurofibromatosis, los sis Serrated Polyposis.

Cov Lus Ceeb Toom thiab Kev Ceev Faj

- Cov neeg mob yuav tsum tsis txhob muab cov qauv yog tias lawv raws plab los sis paub tias muaj ntshav hauv lawv cov zis los sis quav (piv txwv li, los ntawm cov hemorrhoids los ntshav, cov qhov txiav los sis cov qhov txhab los ntshav ntawm lawv txhais tes, los ntshav ntawm qhov quav, los sis los ntshav thaum muaj hli). Yog koj muaj kev los ntshav txawv, koj yuav tsum sab laj nrog koj tus kws kho mob los sis tus kws muab kev kho mob.
- Raws li cov lus qhia hauv teb chaws txog cov hnuv nyoog pom zoo rau kev kuaj mob qog noj ntshav hauv plab hnyuv. Qhov kev txiav txim siab los tshuaj xyuas cov neeg hnuv nyoog tshaj 75 xyoo yuav tsum tau ua raws li tus kheej lub hom phiaj thiab sab laj nrog koj tus kws muab kev kho mob. Cov kev kuaj tau los ntawm cov khoom siv kuaj Cologuard thiab Cologuard Plus yuav tsum tau txhais nrog kev ceev faj rau cov neeg mob laus vim tias kev ntaus nqi ntawm cov kev kuaj tau los uas pom tias muaj kab mob tsis tseeb ntawd yuav nce ntxiv raws lub hnuv nyoog.
- Kev khoom siv kuaj Cologuard thiab Cologuard Plus tej zaum yuav ua rau muaj cov kev kuaj tau los uas pom tias tsis muaj kab mob tsis tseeb los sis kev kuaj tau los uas pom tias muaj kab mob tsis tseeb. Qhov kev kuaj tau los uas pom tias muaj kab mob tsis tseeb yuav tshwm sim thaum cov khoom siv kuaj Cologuard los sis Cologuard Plus qhov kev kuaj tau los pom tias muaj kab mob, tab txawm tias kev siv koob tsum kuaj txoj hnyuv loj yuav tsis pom muaj mob CRC los sis APLs los xij. Qhov kev kuaj tau los tsis pom muaj kab mob tsis



Cologuard, Cologuard Plus, thiab Exact Sciences yog cov cim lag luam ntawm Exact Sciences Corporation. Tag nrho lwm cov cim lag luam yog cov khoom ntawm lawv cov tswv.

© 2023-2025 Exact Sciences Corporation. Tag nrho cov cai raug tseg.

ceeb yog tshwm sim thaum cov khoom siv kuaj Cologuard los sis Cologuard Plus kuaj tsis pom muaj mob APL los sis CRC tab txawm tias thaum kev tsum saib mob hauv txoj hnyuv loj yuav pom ib qho mob txawv txav twg los xij.

- Qhov kev kuaj Cologuard Plus tau qhia pom tias:
- Ntawm txhua 100 leej neeg mob uas kuaj pom tias muaj tus kab mob no, kwv yees li 3 leeg yuav muaj mob CRC, 34 leeg yuav muaj mob APL, 33 leeg yuav muaj mob qog nqaij hlaw uas tsis kis mus rau lwm qhov, thiab 30 leeg yuav tsis pom mob qog nqaij hlaw.
 - Ntawm txhua 10,000 leej neeg mob uas kuaj tsis pom muaj mob qog noj ntshav, kwv yees li 2 leeg yuav ntseeg yuam kev tias lawv tsis muaj mob CRC. Ntawm txhua 100 leej neeg mob uas kuaj pom tias tsis muaj tus kab mob, kwv yees li 7 leej neeg mob yuav raug dag tias lawv tsis muaj APL.

- Qhov kev kuaj tau los ntawm cov koom siv kuaj Cologuard los sis Cologuard Plus uas pom tias tsis muaj kab mob tsis tau txhais tias tsis muaj mob CRC los sis APL. Cov neeg mob uas cov kev kuaj tau los ntawm Cologuard los sis Cologuard Plus tsis muaj kab mob yuav tsum koom nrog cov kev pab cuam kuaj mob qog khees xaws hauv txoj hnyuv loj mus txuas ntxiv, raws li ncau sij hawm uas tau hais qhia kom raug.

- Qhov ua tau zoo xub thawj ntawm qhov kev kuaj Cologuard tau ua pov thawj hauv kev sim ntau zaus kev kawm hla-seem (piv txwv li, kev tshawb fawb ua rau ib lub sij hawm tshwj xeeb). Tseem tsis tau muaj kev tshawb fawb txog kev ua hauj lwm zoo ntawm qhov kev kuaj Cologuard (piv txwv li, cov txiaj ntsig thiab kev pheej hmoo ntawm kev rov ua dua kev sim rau lub sij hawm teev tseg) tau ntsuas nyob hauv txoj kev kawm tshawb fawb mus sij hawm ntev thoob plaws hauv ncau sij hawm peb lub xyos. Tsis tau muaj kev tsim tsa qhov tsis qis dua los sis zoo dua ntawm qhov kev kuaj Cologuard qhov kev cuam tshuam ntawm qhov programmatic piv rau lwm txoj kev tshuaj ntsuam pom zoo rau CRC thiab APL.

- Qhov ua tau zoo ntawm Cologuard Plus tau ua pov thawj hauv kev sim ntau zaus.-Kev kawm hla-seem (piv txwv li, kev tshawb fawb ua rau ib lub sij hawm tshwj xeeb). Tseem tsis tau muaj kev tshawb fawb txog kev ua hauj lwm zoo ntawm Cologuard Plus kev sim (piv txwv li, cov txiaj ntsig thiab kev pheej hmoo ntawm kev rov ua dua kev sim rau lub sij hawm teev tseg). Tsis tau muaj kev tsim tsa qhov tsis qis dua los sis zoo dua ntawm Cologuard Plus qhov kev xeeb qhov kev cuam tshuam ntawm qhov programmatic piv rau lwm txoj kev tshuaj ntsuam pom zoo rau CRC thiab APL.

- **Cov neeg mob yuav tsum pib cov txheej txheem xa rov qab thaum lawv tuaj yeem xa cov khoom cug qauv rov qab mus rau UPS nyob rau tib hnuv uas lawv cug tau qauv los sis hnuv tom qab.**

- Tsis txhob haus cov kua tshuaj tiv thaiv khoom noj kom txhob lwj.
- Tsis txhob cia cov kua tshuaj ntub raug koj daim tawv nqaij los sis ob lub qhov muag.
- Yog tias cov kua tshuaj ntawd ntub raug koj daim tawv nqaij los sis ob lub qhov muag lawm, cia li muab dej los ntshuav thaj chaw ntawd kiag.
- Cov kev pheej hmoo cuam tshuam nrog kev siv cov khoom siv cug qauv yog muaj tsawg, tsis muaj cov xwm txheej tsis zoo loj heev uas tau tshaj tawm hauv cov neeg koom nrog kev sim tshuaj. Cov neeg mob yuav tsum ceev faj thaum qhib thiab kaw lub hau kom tsis txhob muaj kev pheej hmoo ntawm kev mob tes.

Lus Cim: Cov neeg mob tsis tas yuav hloov lawv cov zaub mov noj los sis cov tshuaj rau qhov kev kuaj mob no.

Kev Khaws koj cov khoom siv cug qauv cia

Khaws lub thawv kom ncaj, nyob rau hauv chav sov, deb ntawm tshav ntuj ncaj qha, thiab kom deb ntawm cov menyuam yaus.

Puas muaj lus nug hais txog koj qhov kev kuaj mob?

Yog xav paub ntxiv txog koj qhov kev kuaj mob, ces nkag mus rau hauv koj tus as khauj Exact Sciences ntawm patient.exactsciences.com, los sis tiv tauj rau peb ntawm tus xov toj 1-844-870-8870. Yog tias koj tseem tsis tau muaj ib tus as khauj (account), ces koj tuaj yeem tsim ib tus as kauj tau yooj yim hauv plaub kauj ruam.

Rx only.

EXACT SCIENCES CORPORATION
5505 Endeavor Lane, Madison, WI 53719
www.ExactSciences.com | www.Cologuard.com



1-844-870-8870

LBL-11558_QSG_HMN Rev. 5
09/2025



Cia peb tau txais pib lawm!

Saib koj tus kheej tseem ceeb ua ntej tshaj plaws. Cug koj qhov qauv tau hnuv no.



Luam Theej Duab
Koj tuaj yeem luam
theej duab Lab hav QR
uas yuav pib mus





Hu rau
1-844-870-8870
Muaj ntau hom lus



Sau ntawv sib tham
ntawm Cologuard.com

Npaj koj cov khoom cug qauv

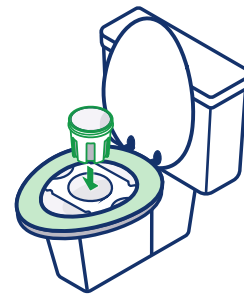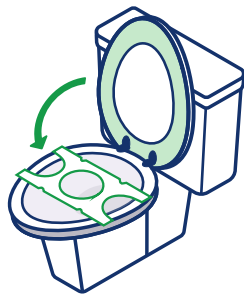

1

- Qhib tag nrho cov khoom uas pom saum toj no. Tsis txhob rub lub hnab yas los sis lub tais dawb tawm ntawm koj cov khoom siv cug qauv.

Yog tias koj raws plab, los ntshav thaum coj khaub ncaws, los sis los ntshav los ntawm cov pob nqaij hlaw hauv ncauj qhov qauv (hemorrhoids), ces tsis txhob cug qhov qauv. Yog tias koj pom muaj ntshav txawv txav, thov hu rau koj tus kws muab kev pab kho mob.


2

- Thov tso zis ua ntej ua ntej yuav nruab qhoov chaw Txawb.
- Tso qhov Chaw Txawb hauv qab lub rooj zaum hauv chav dej.
- Txo lub rooj zaum hauv chav dej.

3

- Tshem lub hau ntawm Lub Thawv Cug Qauv tawm los ntawm kev qhib nws.
- Thaum qhib lawm, muab Lub Thawv tso rau hauv qhov Chaw Txawb.

Sau koj cov qauv quav

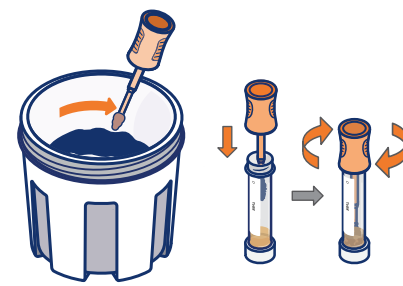


4

- Zaum saum lub chav dej. Tso quav rau hauv ib lub Thawv.
- Koj cov qauv quav tsis tas yuav tsum zoo tag nrho. Saib lub hwj ntim Tshuaj Tiv Thaiv Kom Txhob Lwj hauv koj cov khoom siv. Koj cov quav yuav tsum loj dua lub hau hwj tab sis tsis txhob loj dua lub hwj kiag xwb.

Tsis txhob cia zis nkag mus rau hauv Lub Thawv. Nws TSIS UA CAS yog tias nkag me ntsis xwb. Ua tib zoo tsis txhob pub ntaub siv hauv chav dej poob rau hauv lub Thawv.

5

- Nyws lub Hau tawm ntawm lub Raj. Siv lub Hau los khawb qhov chaw ntawm koj cov quav qauv.

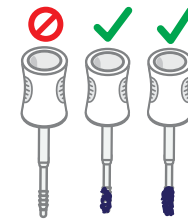


- Npog lub Hau pem ntsis kom ruaj ua ntej muab rov qab rau hauv lub Raj.
- Ntxig lub Hau rau hauv lub Raj thiab ntswj lub hau kom nruj.


Tsis txhob muab cov quav tso rau hauv lub Raj. Tsis txhob muab lub Raj tso rau hauv Lub Thawv Cug Qauv.

6

- Nchuav tag nrho Cov Kua Tshuaj Tiv Thaiv Kom Txhob Lwj hauv lub hwj mus rau hauv Lub Thawv. Yog tias cov quav tsis tau npog tag nrho los tsis UA LI CAS.
- Tshem Lub Thawv tawm ntawm qhov Chaw Txawb. Muab lub hau nyws kaw rov qab rau Lub Thawv kom ceev kiag.

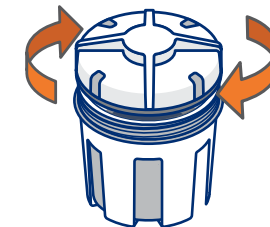


Xa koj cov khoom siv cug qauv

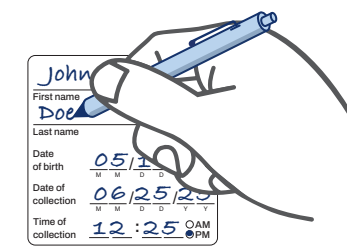


7

- Sau rau ob daim ntawv lo uas muaj nyob rau ntawm Daim Ntawv Sau Lus Qhia Qauv. Muab ib daim tso rau ntawm Lub Thawv thiab ib daim qhwv ncig Lub Raj.

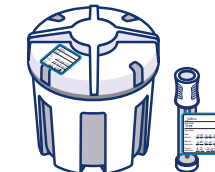


Muab Lub Thawv thiab Lub Raj rov qab rau hauv lub tais dawb hauv Lub Thawv Xa Khoom qub.

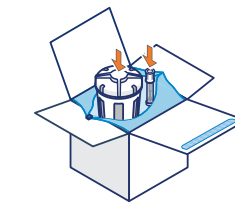

Pov tseg tag nrho lwm yam khoom.

8

- kaw txoj hla sab saum toj ntawm lub hnab kom kaw lub hnab.
- Siv daim nplaum uas lo rau ntawm lub hau ntawm lub thawv los kaw lub thawv.
- Tshem daim ntawv lo rau sab saum toj ntawm daim ntawv xa khoom kom pom daim ntawv lo rau kev xa khoom rov qab uas tau them ua ntej lawm ntawd.

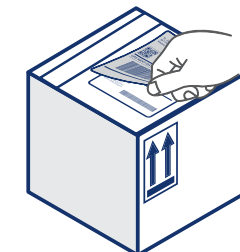

9

- Xa koj cov khoom siv rov qab rau tib hnub ntawd los sis hnub tom qab tom qab tuaj tos hauv ib qho ntawm ob txoj kev.

Cov kev xaiv xa rov qab uas them nyiaj ua ntej:

1. Muab tso rau ntawm UPS.
2. Teem caij rau UPS tuaj tos:
 - Teem sij hawm online ntawm [Cologuard.com/UPS](https://www.cologuard.com/UPS) los sis
 - Hu rau **1-844-870-8870** txhawm rau thov kev pab.

Saib koj cov kev kuaj mob tau nyob rau ntawm: [Cologuard.com/results](https://www.cologuard.com/results)