

Pregled za rak debelog crijeva Cologuard® testom

Cologuard **pokriva** Medicare i većina drugih velikih osiguravajućih društava.

0 USD Većina osiguranih pacijenata plaća **0 USD** za Cologuard test^{1*}

96% Širom zemlje, više od **96%** pacijenata koji koriste Cologuard nema nikakvih troškova za skrining^{1*}



Saznajte više na Cologuard.com/insurance ili pozovite **1-844-870-8870**.
Možemo vam pomoći da postavite prava pitanja vašem osiguravajućem društvu.

Većina osiguravajućih društava **pokriva troškove kolonoskopije** nakon pozitivnog Cologuard testa.^{2,3†}




Video za **pravilnu upotrebu:**
Cologuard.com/use

Cologuard test je **jednostavan** za korištenje kod kuće


Detaljna uputstva se nalaze u Cologuard kutiji.

1 SJEDNITE

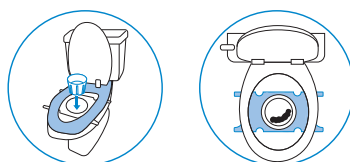


- Koristite Cologuard test kada budete spremni.
- Stavite svoju bijelu posudu za uzorak na WC šolju.
- Sjednite na WC šolju.

2 STRUGANJE

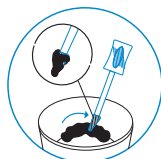


- Ispraznite crijeva u bijelu posudu za uzorak.
- Ostružite stolicu (izmet) štapićem i vratite je u tubu iz kompleta.

3 NATAPANJE I ZATVARANJE

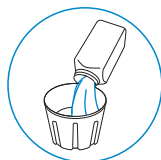

- Napunite posudu za uzorak tekućim konzervansom.
- Zatvorite veliku posudu s uzorkom unutra.

4 ETIKETA I DOSTAVA

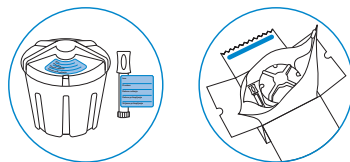

- Popunite obje etikete.
- Vratite ovaj komplet kompaniji Exact Sciences putem UPS®-a.



Vratite svoj uzorak besplatno


1. Zakažite preuzimanje od strane UPS-a na Cologuard.com/UPS ili pozovite **1-844-870-8870**
2. Odnosite svoj komplet u UPS. Pronađite lokalne trgovine i radno vrijeme za preuzimanje na Cologuard.com/UPS

*Nedavna procjena kompanije Exact Sciences je zasnovana na istoriji naplate pacijenata.

†Za većinu pacijenata, ovo će stupiti na snagu 1. januara 2023. i ne odnosi se na komercijalne planove sa statusom izuzeća.^{2,3}

Rezultati vašeg Cologuard® testa

Vaš ljekar bi trebao imati rezultate dostupne za otprilike 8-10 dana nakon što Exact Sciences primi vaš uzorak.

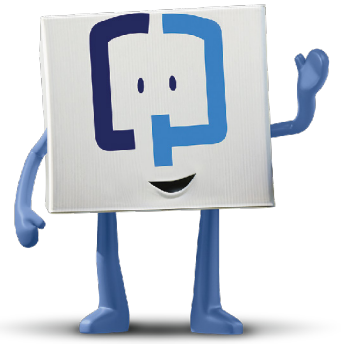

Cologuard test traži **promijenjenu DNK** i **krv** (znakove raka) u vašoj stolici.⁴



Negativan rezultat znači da Cologuard test nije pronašao moguće znakove raka. Najvjerovatnije nemate kolorektalni karcinom.⁴



Pozitivan rezultat znači da je Cologuard test pronašao moguće znakove raka. Pozitivan rezultat **ne znači** da imate rak.⁴ Zakažite kontrolni pregled kod svog ljekara.

Neke osobe dobiju lažno (netačne) pozitivne i negativne rezultate. Uvijek pitajte svog ljekara o sljedećim koracima za skrining.⁴

Više informacija i videozapisa pronađite na:
[Cologuard.com/UnderstandingResults](https://cologuard.com/UnderstandingResults)

Indikacije i važne informacije o riziku

Cologuard® i Cologuard Plus® testovi su namijenjeni skriningu odraslih osoba starijih od 45 godina koje imaju prosječan rizik od kolorektalnog karcinoma, otkrivanjem određenih DNK markera i krvi u stolici. Ne koristite Cologuard proizvode ako ste imali ličnu anamnezu kolorektalnog karcinoma ili prekanceroznih polipa, imate inflamatornu bolest crijeva, porodičnu anamnezu kolorektalnog karcinoma, određene nasljedne sindrome ili pozitivan rezultat druge metode skrininga raka debelog crijeva unutar preporučenog intervala skrininga za taj test. Razgovarajte sa svojim ljekarom ako se bilo koja od ovih situacija odnosi na vas.

Rezultate Cologuard testova treba tumačiti s oprezom. Pozitivan rezultat testa ne potvrđuje prisustvo raka. Pacijente s pozitivnim rezultatom testa treba uputiti na kolonoskopiju. Negativan rezultat testa ne potvrđuje odsustvo raka. Pacijenti s negativnim rezultatom testa trebaju razgovarati sa svojim ljekarom o tome kada im je potrebno ponovo se testirati. Mogu se pojaviti lažno pozitivni i lažno negativni rezultati. Samo na recept.

Reference: 1. Internal Data on File. Exact Sciences. 2. FAQs about Affordable Care Act Implementation Part 51, Families First Coronavirus Response Act and Coronavirus Aid, Relief, and Economic Security Act Implementation. US Department of Health & Human Services. Published January 10, 2022. Accessed March 30, 2026. <https://www.hhs.gov/guidance/document/faqs-about-affordable-care-act-implementation-part-51>. 3. CMS. Calendar year (CY) 2023 Medicare physician fee schedule final rule. Published November 1, 2022. Accessed March 30, 2026. <https://www.cms.gov/newsroom/fact-sheets/calendar-year-cy-2023-medicare-physician-fee-schedule-final-rule>. 4. Exact Sciences. 2026. Cologuard Patient Brochure.