

Screen for colon cancer with the Cologuard[®] test

The Cologuard test is **covered** by Medicare and most major insurers.

\$0

Most insured patients pay **\$0** for the Cologuard test^{1*}

96%

Nationwide, more than **96%** of Cologuard patients have no out-of-pocket cost for screening^{1*}



Learn more at Cologuard.com/insurance or call **1-844-870-8870**.
We can help you ask your insurance company the right questions.

Most insurers **cover a follow-up colonoscopy** after a positive Cologuard screening.^{2,3†}




How To Use video:
Cologuard.com/use

The Cologuard test is **easy** to use at home

Full Detailed Instructions are in the Cologuard Box

1


SIT

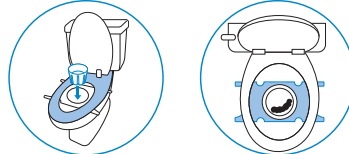

- Use the Cologuard test when you are ready.
- Put your white sample container on the toilet.
- Sit on the toilet.

2


SCRAPE

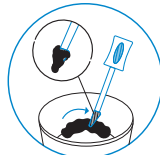

- Poop into the white sample container.
- Scrape your stool (poop) with stick and place back into the tube from the kit.

3


SOAK & SEAL

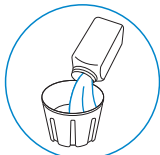

- Fill the sample container with the liquid preservative.
- Close the large container with your sample inside.

4

LABEL & SHIP

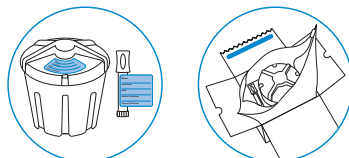

- Fill out both labels.
- Return this kit to Exact Sciences through UPS[®].



Return your sample for free


1. Schedule a UPS pickup at Cologuard.com/UPS or call **1-844-870-8870**
2. Take your kit to UPS. Find local stores and drop-off hours at Cologuard.com/UPS

^{*}Exact Sciences recent estimate based on historical patient billing.

[†]For most patients, this will take effect on January 1, 2023, and does not apply to commercial plans with exempted status.^{2,3}

Your Cologuard® test results

Your healthcare provider should have your results available in about 8-10 days after Exact Sciences receives your sample.

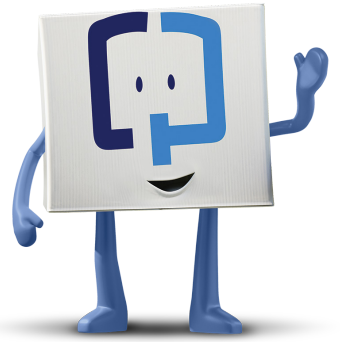

The Cologuard test looks for **altered DNA** and **blood** (signs of cancer) in your stool⁴

-  A negative result means the Cologuard test did not find possible signs of cancer. You most likely do not have colorectal cancer.⁴
-  A positive result means the Cologuard test found possible signs of cancer. A positive result does **not** mean you have cancer.⁴ Schedule a follow-up appointment with your healthcare provider.

Some people get false (incorrect) positive and negative results. Always ask your healthcare provider about next steps for screening.⁴

Find more information and videos at:
[Cologuard.com/UnderstandingResults](https://cologuard.com/UnderstandingResults)

Indications and Important Risk Information

The Cologuard test is intended to screen adults 45 years of age and older who are at average risk for colorectal cancer by detecting certain DNA markers and blood in the stool. Do not use if you have had adenomas, have inflammatory bowel disease and certain hereditary syndromes, or a personal or family history of colorectal cancer. A Cologuard test is not a replacement for colonoscopy in high risk patients. Cologuard test performance in adults ages 45-49 is estimated based on a large clinical study of patients 50 and older.

The Cologuard test result should be interpreted with caution. A positive test result does not confirm the presence of cancer. Patients with a positive test result should be referred for colonoscopy. A negative test result does not confirm the absence of cancer. Patients with a negative test result should discuss with their doctor when they need to be tested again. False positives and false negative results can occur. In a clinical study, 13% of people without cancer received a positive result (false positive) and 8% of people with cancer received a negative result (false negative). Rx only

References: 1. Internal Data on File. Exact Sciences. 2. FAQs about Affordable Care Act Implementation Part 51, Families First Coronavirus Response Act and Coronavirus Aid, Relief, and Economic Security Act Implementation. US Department of Health & Human Services. Published January 10, 2022. Accessed May 22, 2025. <https://www.hhs.gov/guidance/document/faqs-about-affordable-care-act-implementation-part-51>. 3. CMS. Calendar year (CY) 2023 Medicare physician fee schedule final rule. Published November 1, 2022. Accessed May 22, 2025. <https://www.cms.gov/newsroom/fact-sheets/calendar-year-cy-2023-medicare-physicianfee-schedule-final-rule>. 4. Exact Sciences. 2024. Cologuard Patient Brochure.

EXACT SCIENCES CORPORATION | 5505 Endeavor Lane, Madison, WI 53719
ExactSciences.com | ExactLabs.com | 1-844-870-8870

Cologuard and Exact Sciences are registered trademarks of Exact Sciences Corporation. All other trademarks are the properties of their respective owners.

© 2025 Exact Sciences Corporation. All rights reserved. M-US-CG-05164_LL_May 2025

