

Kuaj kaab mob khees xaws txuj nyuv laug kws siv kev kuaj Cologuard®

Kev kuaj Cologuard yog ib yaam kws Medicare hab feem ntau ntawm tej pab kas phais **them**.

\$0

Feem coob cov tuab neeg kws muaj pab kas phais them **\$0** rua qhov kev kuaj Cologuard^{1*}

96%

Thoob teb chaws, ntau tshaaj le **96%** ntawm cov tuab neeg mob kws siv Cologuard tsis raug them nyaj huv naab nyaj rua qhov kev kuaj nuav^{1*}



Moog kawm ntxiv ntawm Cologuard.com/insurance los yog hu rua **1-844-870-8870**.

Peb yuav paab tau koj hu koj tug pab kas phais kuam tau cov lug noog kws raug.

Feem ntau cov pab kas phais **them** rua kev soj lw taug qaab saib kev **xoo txuj nyuv laug** tom qaab kev kuaj Cologuard pum tas muaj mob.^{2,3†}




Qhov kev kuaj Cologuard siv tau **yooj yim** tom tsev

Cov Lug Taw Qha Kws Meej Muaj nyob rua huv lub Thawv Cologuard

Dlaim yeeb yaam **Qha Kev Siv**: Cologuard.com/use

1


ZAUM QUAS TSAWG

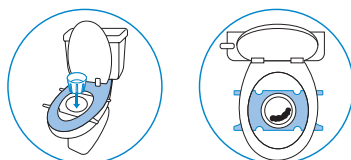

- Siv qhov kev kuaj Cologuard thaum koj npaaj txhij.
- Muab koj lub taub yaas khaws qhov qauv xim dlawb tso rua huv lub chaw zaum tso quav.
- Zaum quas tsawg rua ntawm lub chaw zaum tso quav.

2


DLIG

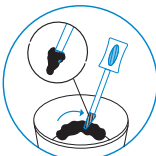

- Tso quav rua huv lub taub yaas khaws qauv xim dlawb.
- Muab tug paas coj lug dlig koj ib qho quav tso rua huv lub taub yaas hab muab tso rov qaab rua huv pob khoom siv.

3


TSAU DLEJ HAB KAW KUAM KHOV KHO

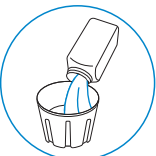

- Laub cov kua tshuaj kuam txhob lwj kuam puv lub taub yaas ntim quav.
- Muab lub taub yaas luj kws ntim koj qhov quav nyob rua huv kaw.

4

LU DLAIM NTAWV HAB XAA MOOG

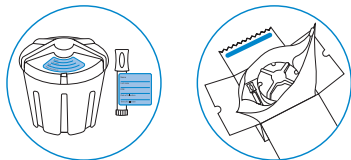

- Sau ob dlaim ntawv lu tuab si.
- Xaa pob khoom nuav rov qaab moog rua Exact Sciences lug ntawm UPS®.



Xaa koj pob khoom rov qaab yam tsis tau them nge dlaab tsi


1. Teem sij hawm rua UPS tuaj nqaa ntawm koj tsev ntawm Cologuard.com/UPS los yog hu rua **1-844-870-8870**
2. Nqaa koj pob khoom moog rua tim UPS. Nrhav cov tsev xaa ntawv hab sij hawm xaa khoom ntawm Cologuard.com/UPS

^{*}Exact Sciences cov kev kwv yeeg tsis ntev lug nuav raws le tej nuj nqes kws tug tuab neeg mob them yaav dlhau lug.

[†]Rua cov tuab neeg mob feem coob, qhov nuav yuav pib siv rua Lub Ib hlis ntuj tim 1, 2023, hab tsis siv rua cov kev npaaj rua kev lag luam kws muaj qhov xwm txhej tau txais kev zaam.^{2,3}

Koj qhov kev kuaj Cologuard® pum tawm lug

Koj tug kws khu mob yuav tau txais koj cov kev kuaj le ntawm 2 lub lim tam tom qaab kws Exact Science txais tau koj le qauv kuaj.

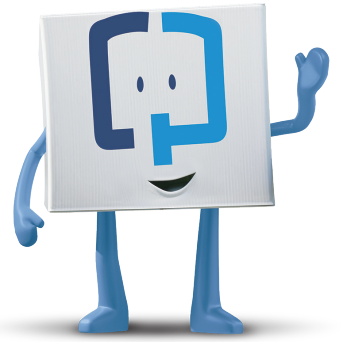

Kev kuaj Cologuard yuav kuaj nrhav **DNA** hab **ntshaav kws muaj kev hloov pauv moog** (cov cim khees-xaws) huv koj cov quav⁴

- Yog kuaj tawm lug tas tsis muaj mob ces txhais tau tas kev kuaj Cologuard tsis pum cov cim kaab mob khees-xaws. Feem ntau koj yuav tsis muaj mob khees-xaws huav txuj nyuv laug.⁴
- + Yog kuaj tawm lug tas muaj mob tes txhais tau tas kev kuaj Cologuard pum muaj cov cim kaab mob khees-xaws. Yog kuaj tawm lug tas muaj mob kuj **tsis** tau txhais kag tas koj muaj tug mob khees-xaws.⁴ Teem caij moog ntsib nrug koj tug kws khu mob.

Qee tug tuab neeg muaj peev xwm kuaj pum tuas muaj mob hab pum tas tsis muaj mob cuav (tsis muaj tseeb). Txhua zag ncu ntsoov noog koj tug kws khu mob txug cov kauj ruam yuav kuaj mob tom ntej seb muaj le caag.⁴

Nrhav lug qha hab cov yeeb yaaj kab qha rua ntawm: **Cologuard.com/UnderstandingResults**

Cov Kev Qha hab Ntaub Ntawv Kev Pheej Hmoo Kws Tseem Ceeb

Kev kuaj Cologuard yog siv rua cov tuab neeg laug nub nyoog 45 xyoos rov sau kws muaj feem pes nraab tau tug kaab mob khees-xaws txuj nyuv laug lug ntawm kev kuaj nrhav cov cim DNA (DNA markers) qee hom hab cov ntshaav huv cov quav. Tsis txhob siv yog tas koj cov nyuv muaj nqaj hlaav (adenomas), koj muaj mob nyuv o, hab lwm yaam mob raws caaj ceg, los yog tas koj tug kheej los yog tsev tuab neeg muaj mob khees-xaws txuj hnyuv laug. Kev kuaj Cologuard tsis yog ib qho hloov tau txuj kev moog tsom xoos plaab nyuv rua cov tuab neeg mob kws muaj kev pheej hmoo sab. Qhov txaj ntsig zoo rua kev kuaj Cologuard rua cov tuab neeg nub nyoog 45-49 xyoo yog kwv yeeg raws le kev kawm fab kev khu mob luj huv cov tuab neeg mob kws muaj nub nyoog 50 xyoo rov sau.

Yuav tsum tau ua tuab zoo saib cov kev ntsuag kws siv Cologuard lug kuaj. Txawm tas kuaj pum tau tas muaj mob los tsis txhais tau tas muaj tug mob khees-xaws nyob huv. Cov tuab neeg mob kws ntsuag tau tas muaj yuav tsum tau raug xaa moog tsom xoos cov plaab nyuv. Txawm tas kuaj pum tau tas tsis muaj mob los tsis tau txais tau tas tsis muaj tug mob khees-xaws nyob huv. Cov tuab neeg mob kws kuaj pum tau tas tsis muaj mob yuav tsum nrug puab tug kws khu mob thaam txug tas thaum twg puab yuav tsum rov kuaj dlua. Cov kev kuaj nuav muaj peev xwm kuaj tau tas muaj mob cuav hab tsis muaj mob cuav hab. Ib txuj kev tshawb fawb pum tas 13% ntawm cov tuab neeg kws tsis muaj mob khees-xaws kuaj pum tas muaj mob cuav (muaj mob tsis raug) hab 8% ntawm cov tuab neeg kws muaj mob khees-xaws kuaj pum tas tsis muaj mob cuav (tsis muaj mob tsis raug). Rx xwb

Cov ntawv qha: 1. Internal Data on File. Exact Sciences. 2. FAQs about Affordable Care Act Implementation Part 51, Families First Coronavirus Response Act and Coronavirus Aid, Relief, and Economic Security Act Implementation. US Department of Health & Human Services. Tau Luam Tawm Thaum Lub Ib Hlis Ntuj Tim 10, 2022. Nkaag Moog Tau Thaum Lub Tsib Hlis Ntuj Tim 2, 2024. <https://www.hhs.gov/guidance/document/faqs-about-affordable-care-act-implementation-part-51>. 3. CMS. Calendar year (CY) 2023 Medicare physician fee schedule final rule. Tau Luam Tawm Nyob Rau Hauv Kaum Ib Hlis 1, 2022. Nkaag Moog Tau Thaum Lub Tsib Hlis Ntuj Tim 2, 2024. <https://www.cms.gov/newsroom/fact-sheets/calendar-year-cy-2023-medicare-physicianfee-schedule-final-rule>. 4. Exact Sciences. 2022. Dlain Ntawv Quav Qha Rua Tug Tuab Neeg Mob Ntawm Cologuard.